



Product information

Texmax 2 layer cotton face mask

Features:

Can be washed at 90°C
Must be washed in a washing bag
Chemical cleaning may be used
Can be dried in a dryer
Can be ironed at 240°C
Can be used multiple times
Materials: healthcare-grade textile (100% cotton), thread, rubber
The mask can be shaped to the form of the nose using the built in wire
Produced in Hungary

How to use:**How to put on face mask:**

The mask must be washed at 90°C before use
Clean your hands with soap and water or hand sanitizer before touching the mask
Make sure there are no obvious tears or holes in either side of the mask
Determine which side of the mask is the top
Determine which side of the mask is the front
Hold the mask by the ear loops. Place a loop around each ear
Mold or pinch the stiff edge to the shape of your nose

How to remove face mask:

Clean your hands with soap and water or hand sanitizer before touching the mask
Avoid touching the front of the mask as the front of the mask is contaminated
Only touch the ear loops
Hold both of the ear loops and gently lift and remove the mask
Clean your hands with soap and water or hand sanitizer
The mask must be washed at 90°C after use
The filters can not be washed. Before washing the mask, remove the filter.

PM2.5 Filter

Texmax face masks can be bought with the preparation for PM2.5 filters. These filters are available at the webshop and can be bought with the masks or separately.

Texmax 2 layer poliester and cotton face mask

Features:

Can be washed at 60°C
Must be washed in a washing bag
Chemical cleaning can not be used
Can not be dried in a dryer
Can be ironed at 150°C
Can be used multiple times
Materials: healthcare-grade textile (100% cotton), polyester, thread, rubber
The mask can be shaped to the form of the nose using the built in wire
Produced in Hungary

How to use:**How to put on face mask:**

The mask must be washed at 60°C before use
Clean your hands with soap and water or hand sanitizer before touching the mask
Make sure there are no obvious tears or holes in either side of the mask
Determine which side of the mask is the top
Determine which side of the mask is the front
Hold the mask by the ear loops. Place a loop around each ear
Mold or pinch the stiff edge to the shape of your nose

How to remove face mask:

Clean your hands with soap and water or hand sanitizer before touching the mask
Avoid touching the front of the mask as the front of the mask is contaminated
Only touch the ear loops
Hold both of the ear loops and gently lift and remove the mask
Clean your hands with soap and water or hand sanitizer
The mask must be washed at 60°C after use
The filters can not be washed. Before washing the mask, remove the filter.

PM2.5 Filter

Texmax face masks can be bought with the preparation for PM2.5 filters. These filters are available at the webshop and can be bought with the masks or separately.

Manufacturing information

The 2 layer face mask has an outer layer made of polyester and an inner layer made of 100% healthcare-grade cotton textile.
The cotton part must face the mouth when in use.



PM2.5 Filter

The PM2.5 filter purifies air, it filters out particles bigger than 2.5 microns like smog and pollution. These are the particles that can make you cough and sneeze in bad polluted city air. The filter does not let these particles into your lung.

The PM2.5 filter uses an activated carbon filtration system so in time it fills with particles. The PM2.5 filters must be changed in the masks after 40 hours of use. The filters can not be washed. Before washing the mask, remove the filter.



5 LAYERS PM 2.5 FILTER

